

www
HOGGI®.de

Made in Germany
by HOGGI 

RTW

Rock The World.

Growing foldable wheelchair for challenging environments.





Setting and assembly manual.

Index



1	Settings	3-13
1.1	Seat width cross brace	3
1.2	Seat depth by selecting cross brace	4
1.3	Seat depth by moving cross brace size 1-30	5
1.4	Seat depth by moving cross brace size 1-35	6
1.5	Seat depth by moving cross brace size 1-40	7
1.6	Lower leg length	8
1.7	Back height	9
1.8	Handle / back cane height	10
1.9	Rear wheels	11
1.10	Front casters	12
1.11	Active degree	13
2	Sub assemblies	14-19
2.1	Guide lever	14
2.2	Cross brace bushing	15
2.3	Cross brace	16/17
2.4	Footrest, left	18
2.5	Footrest, right	19
3	Final assemblies	20-29
3.1	Base frame / side frame	20
3.2	Adapter plate / fender bracket	21
3.3	Back cane / back sling	22
3.4a	Front caster, 190 x 50 mm	23
3.4b	Front caster, 140 x 40 mm	24
3.5	Footrest	25
3.6	Side guards	26
3.7	Rear wheels	27
3.8	Wheel locks	28
3.9	Seat sling	29
4	Your notes	30-31

Seat width 30 cm (12")

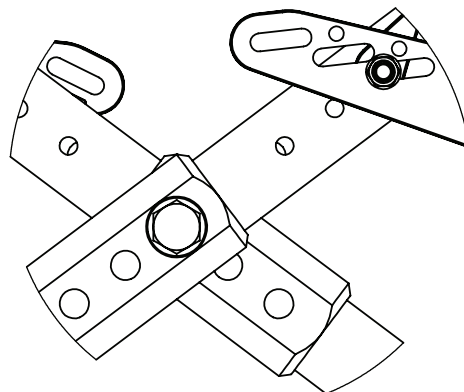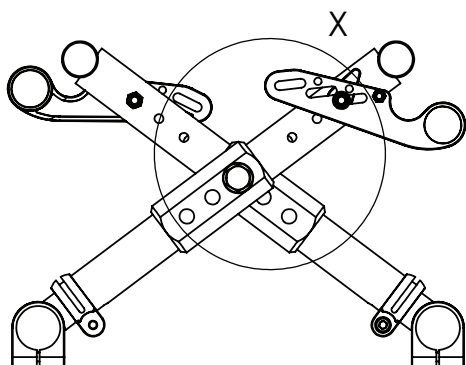



X (2 : 5)

Seat width 35 cm (14")

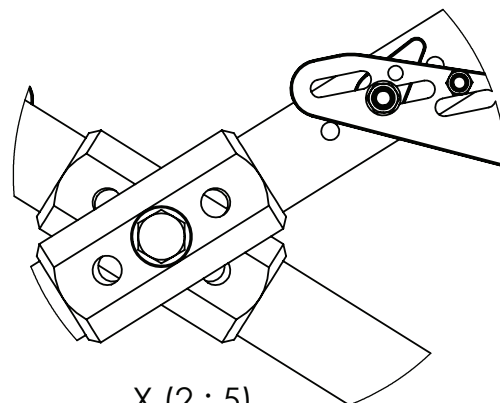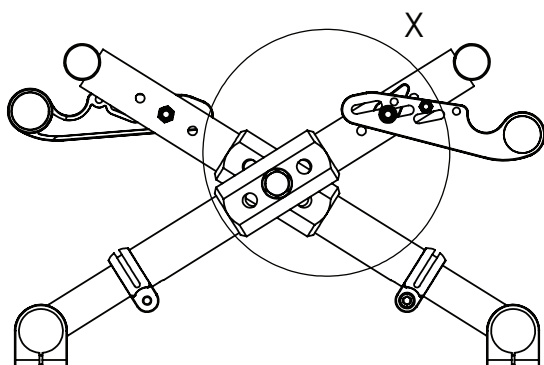


X (2 : 5)

Seat width 40 cm (16")

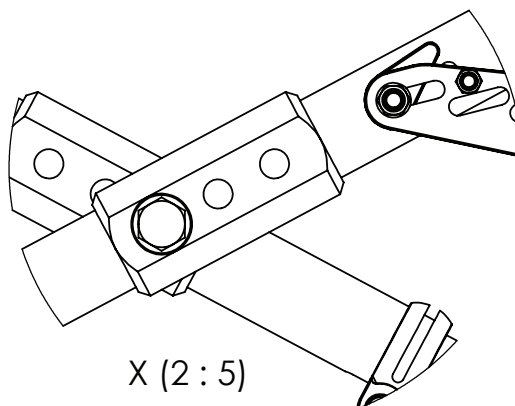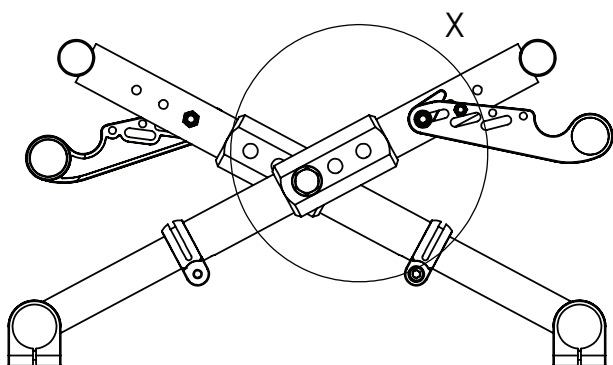


X (2 : 5)



3225-1220
Cross brace top size 1
SD 30 cm (12")

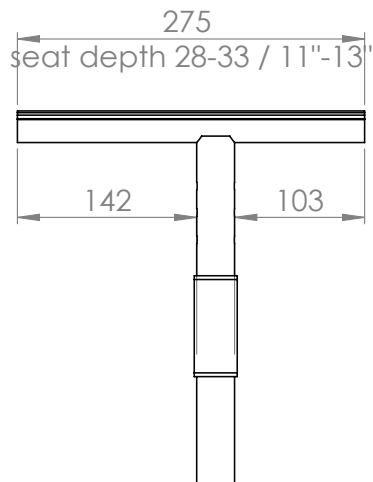

3225-1230
Cross brace top size 1
SD 35 cm (14")



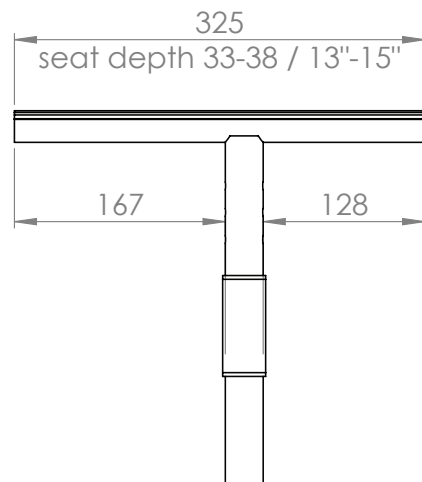


3225-1240
cross brace top size 1
SD 40 cm (16")

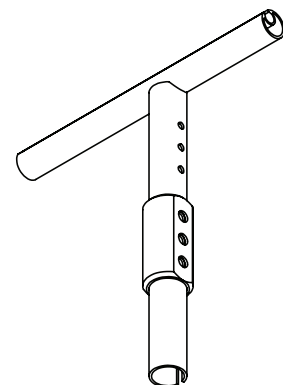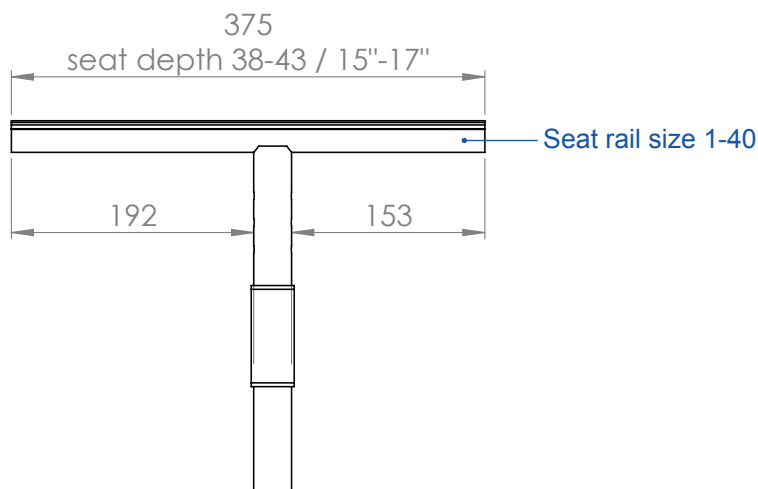




Cross brace tops sizes 30 and 35 cm could be made by cutting seat rail of cross brace top size 1-SD 40

3225-1220 Cross brace size 1-30

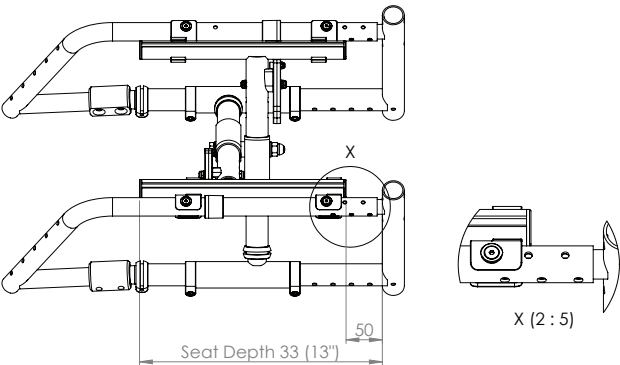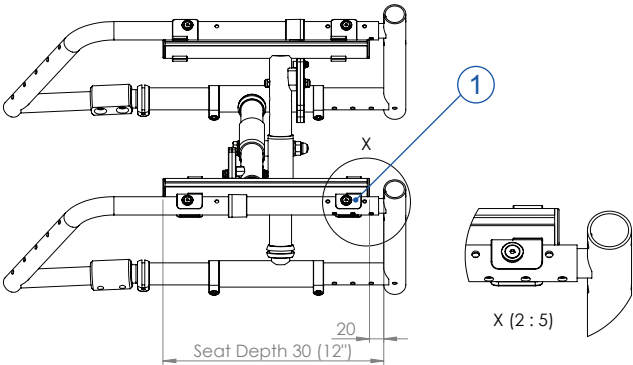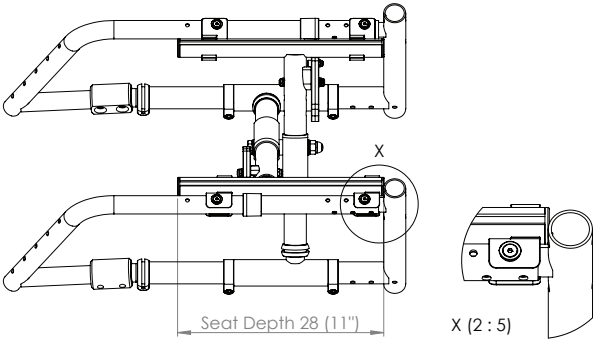


Position	Article number	Article description	Quantity
1	3224-1292	Seat rail rest	4 Pc.

3225-1230 Cross brace size 1-35

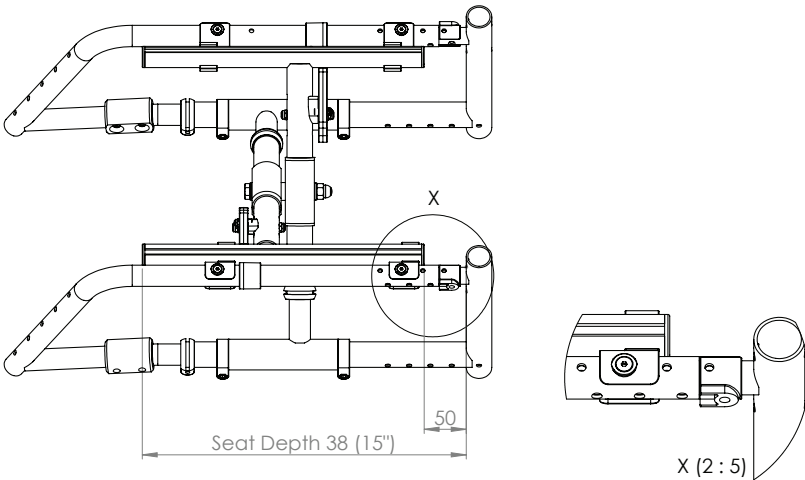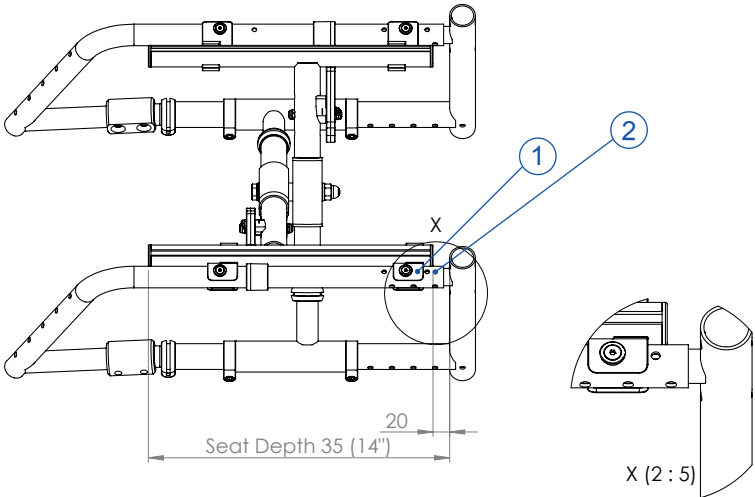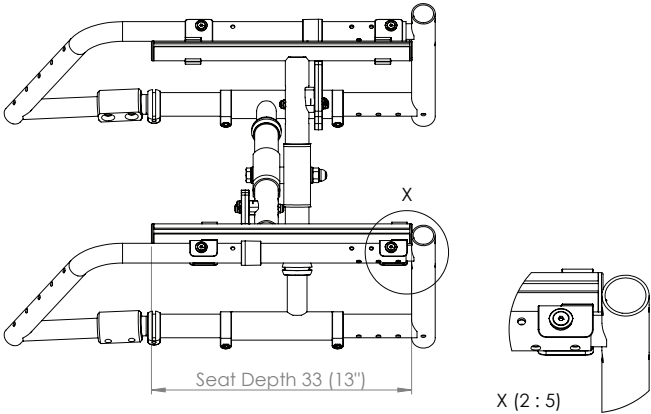



Position	Article number	Article description	Quantity
1	3224-1292	Seat rail rest	4 Pc.
2	3224-1152	Spacer 25 / 50	2 Pc.

3225-1240 Cross brace size 1-40

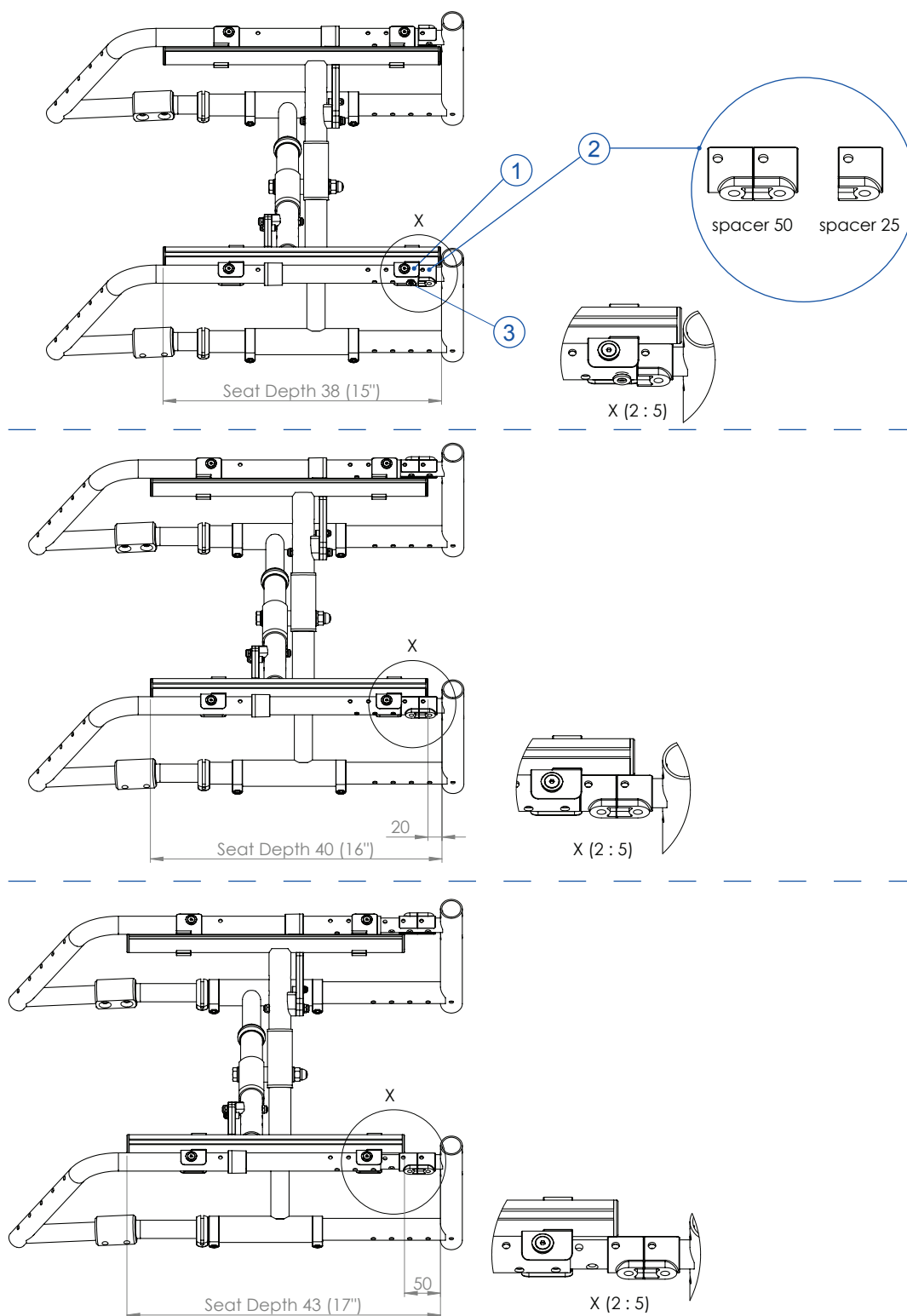



Position	Article number	Article description	Quantity
1	3224-1292	Seat rail rest	4 Pc.
2	3224-1152	Spacer 25 / 50	2 Pc.
3	1051-0006	Washer zp, DIN/ISO 125/7089, A6,4	4 Pc.

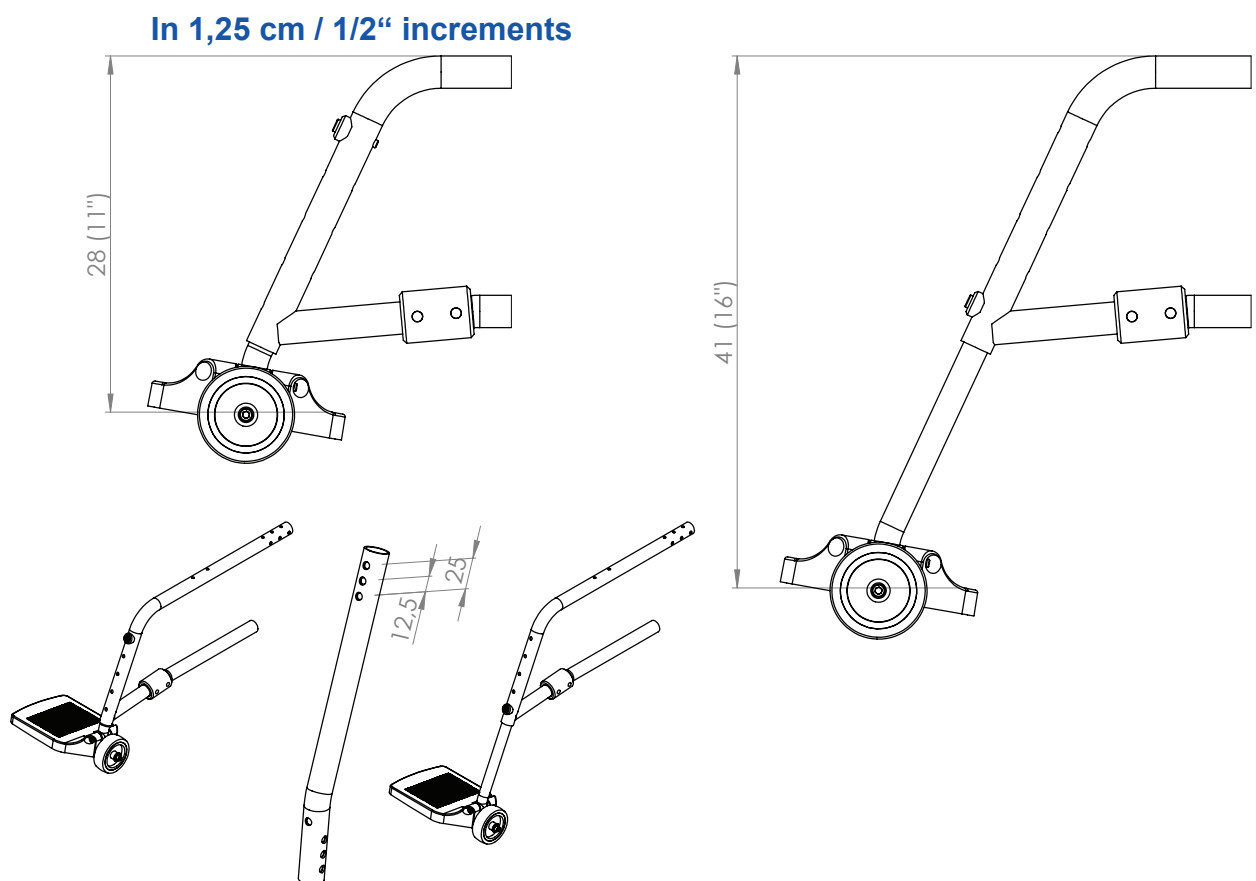

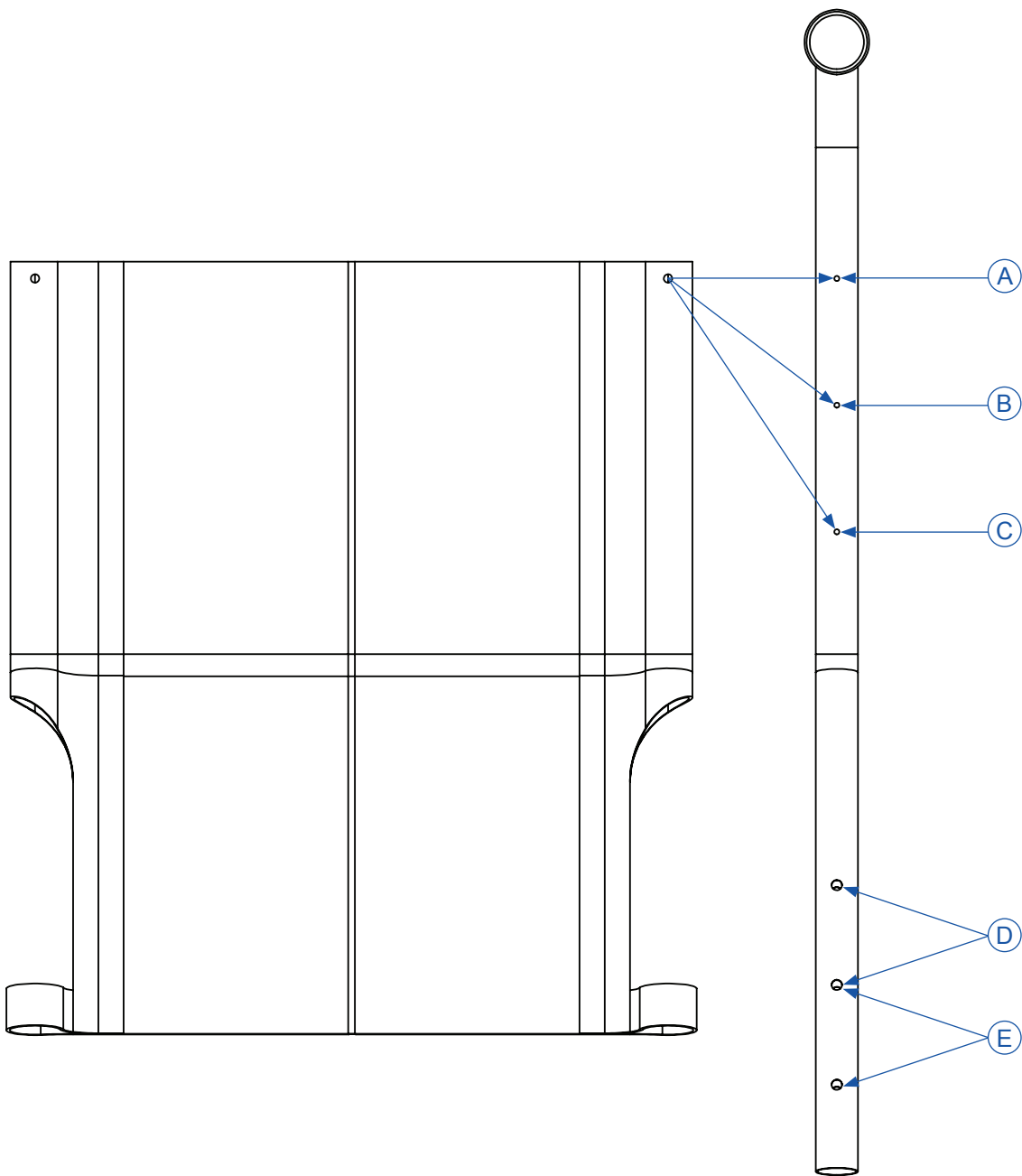



Position	D (handle height 86 cm)	E (handle height 92 cm)
A	39 cm / 15,5"	45 cm / 18"
B	31,5 cm / 12,5"	37,5 cm / 15"
C		30 cm / 12"

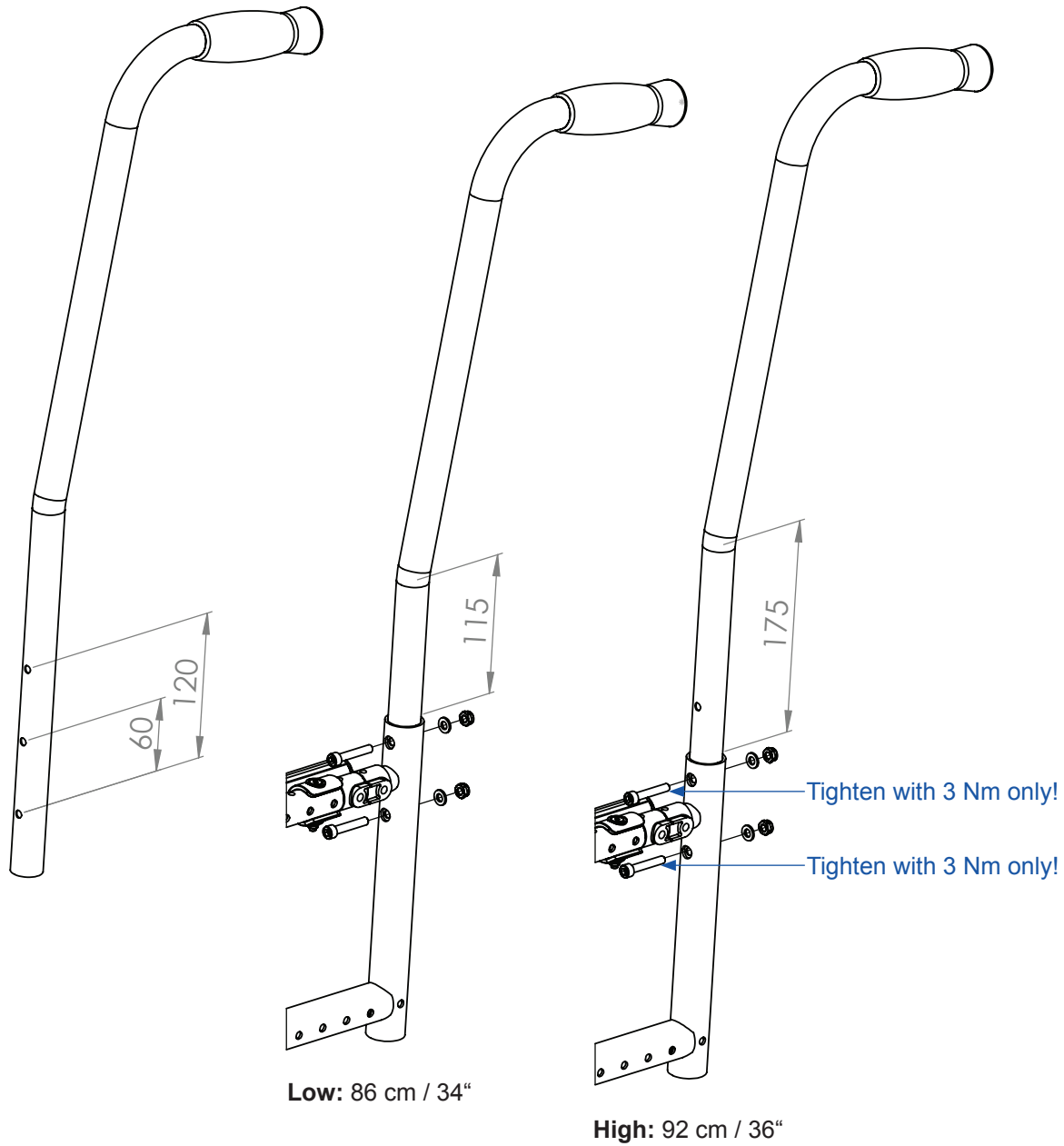

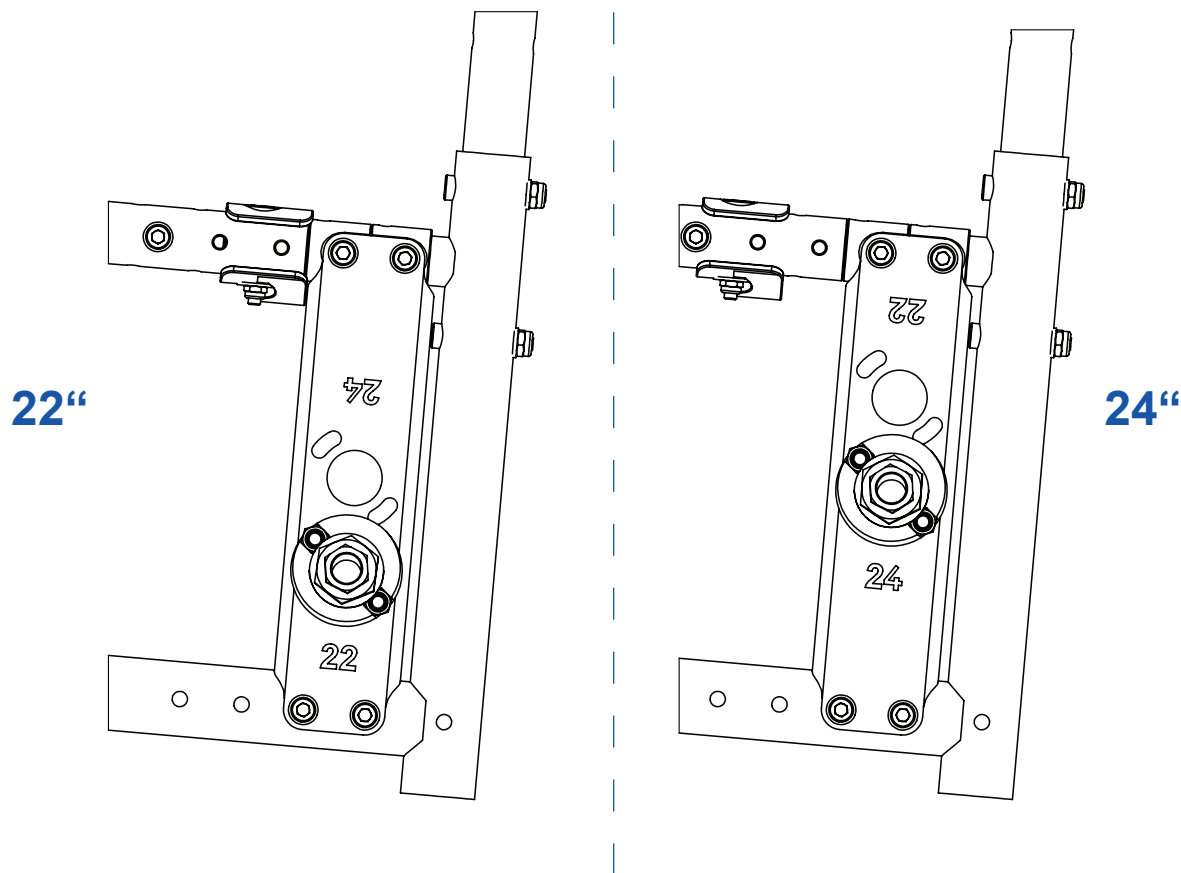

Adjustment chamber adapter

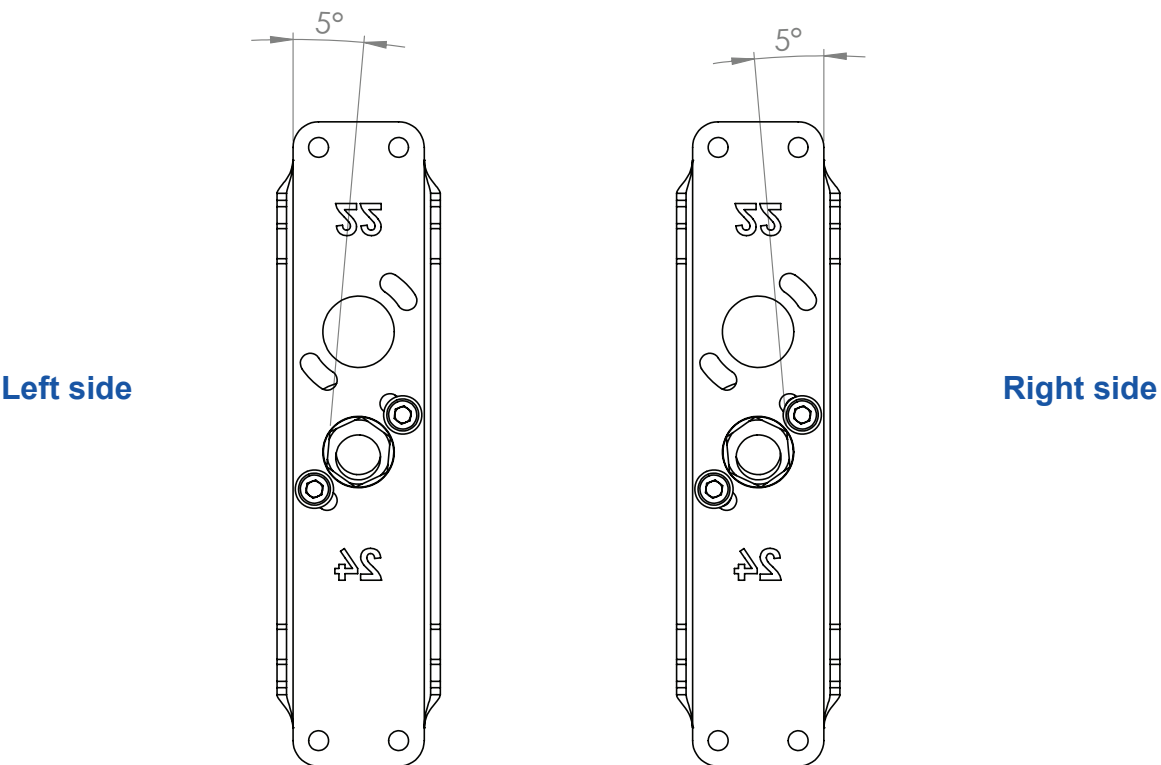


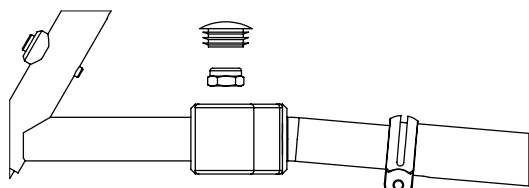


3224-1176-007
Caster fork bushing, **long**



Hexagon head screw zp, 931/4014
M12x95, SW19

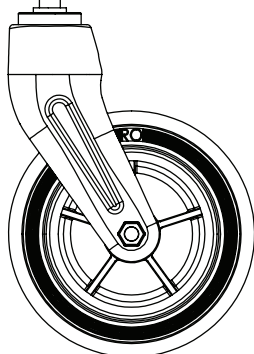


140 mm x 40 mm / 5,5" x 1,6"

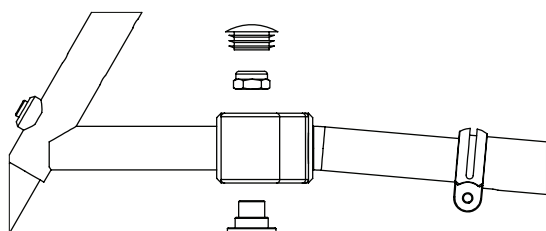


3224-1173
Caster fork bushing




Hexagon head screw zp, 931/4014
M12x75, SW19

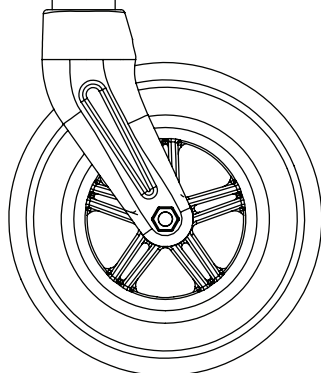


190 mm x 50 mm / 7,5" x 2"

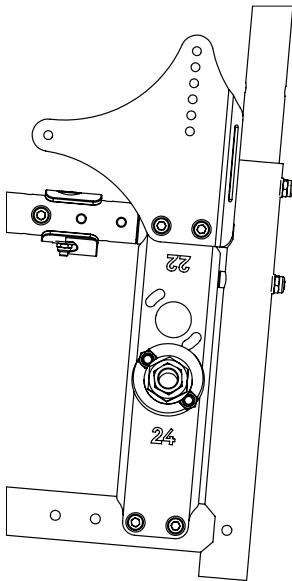




Active degree 1

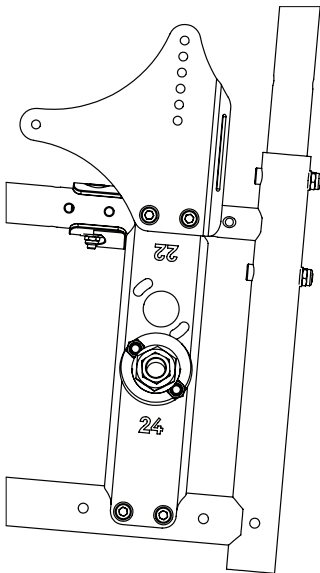


Active degree 2

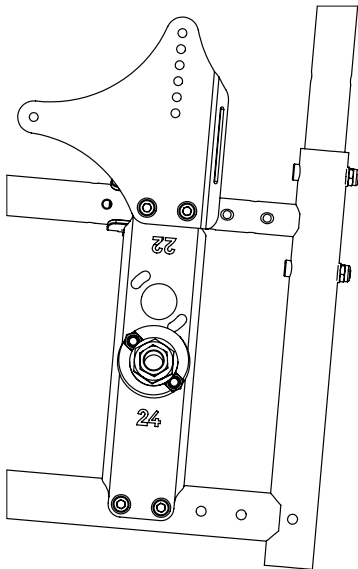


Active degree 3

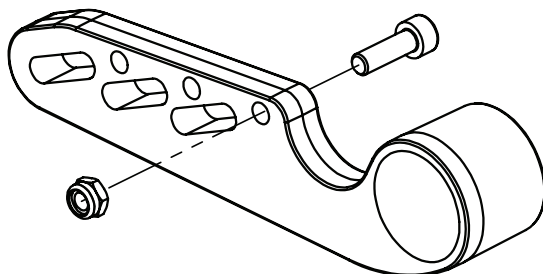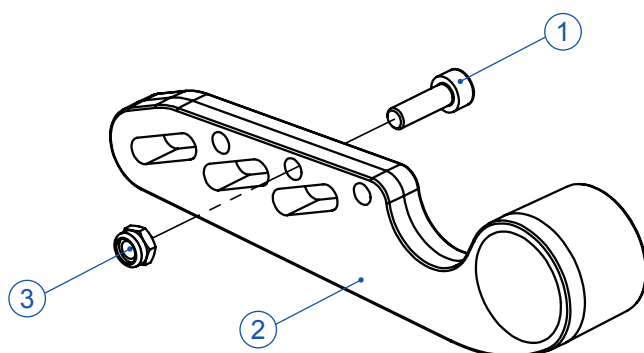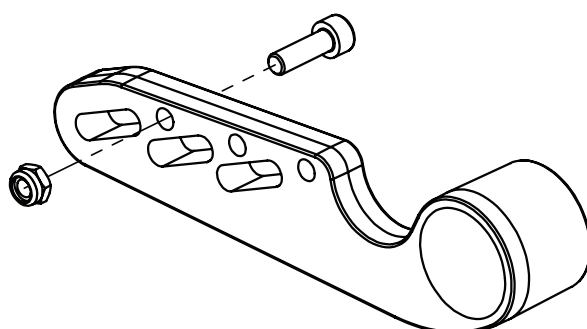**Seat width 30****Seat width 35****Seat width 40**

Position	Article number	Article description	Quantity
1	1021-0516	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M5x16	2 Pc.
2	3224-1259	Guide lever	2 Pc.
3	1042-0005	Hexagon nut self-locking zp, DIN/ISO 985, M5	2 Pc.

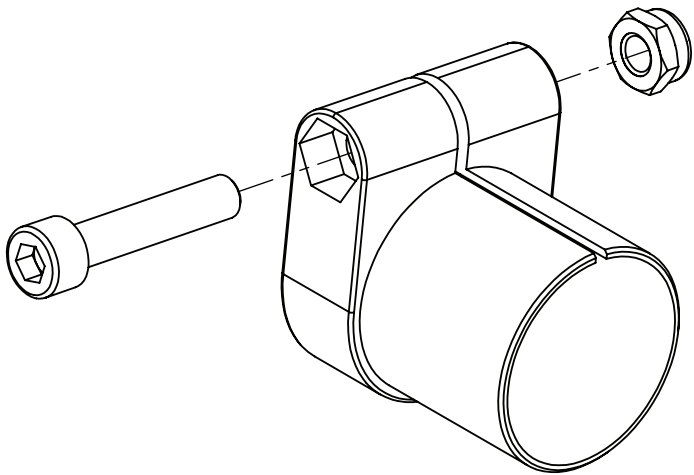


2 x Left side

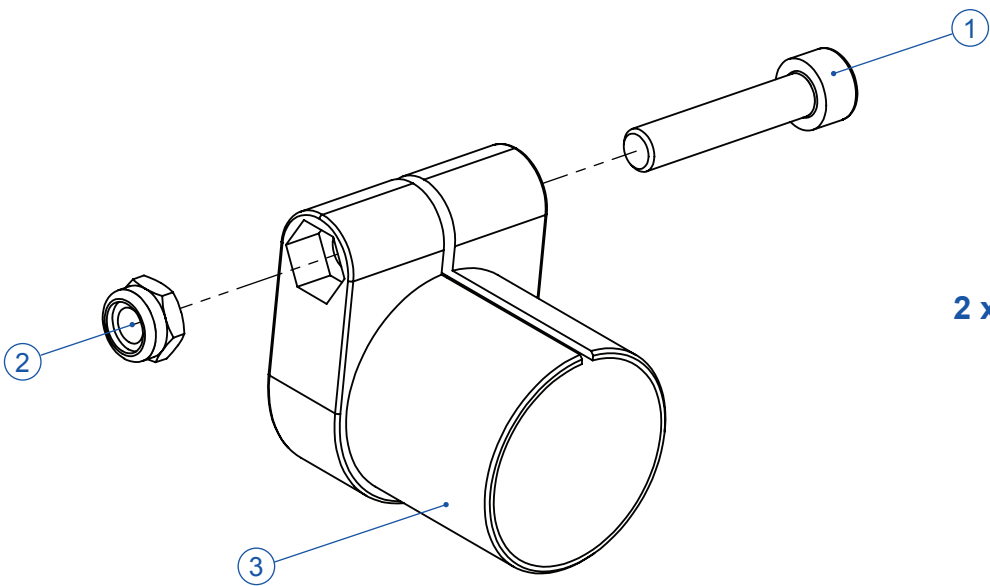


2 x Right side


Position	Article number	Article description	Quantity
1	1021-0630	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x30	4 Pc.
2	1042-0006	Hexagon nut self-locking zp, DIN/ISO 985, M6	4 Pc.
3	3224-1293	Cross brace bushing	4 Pc.

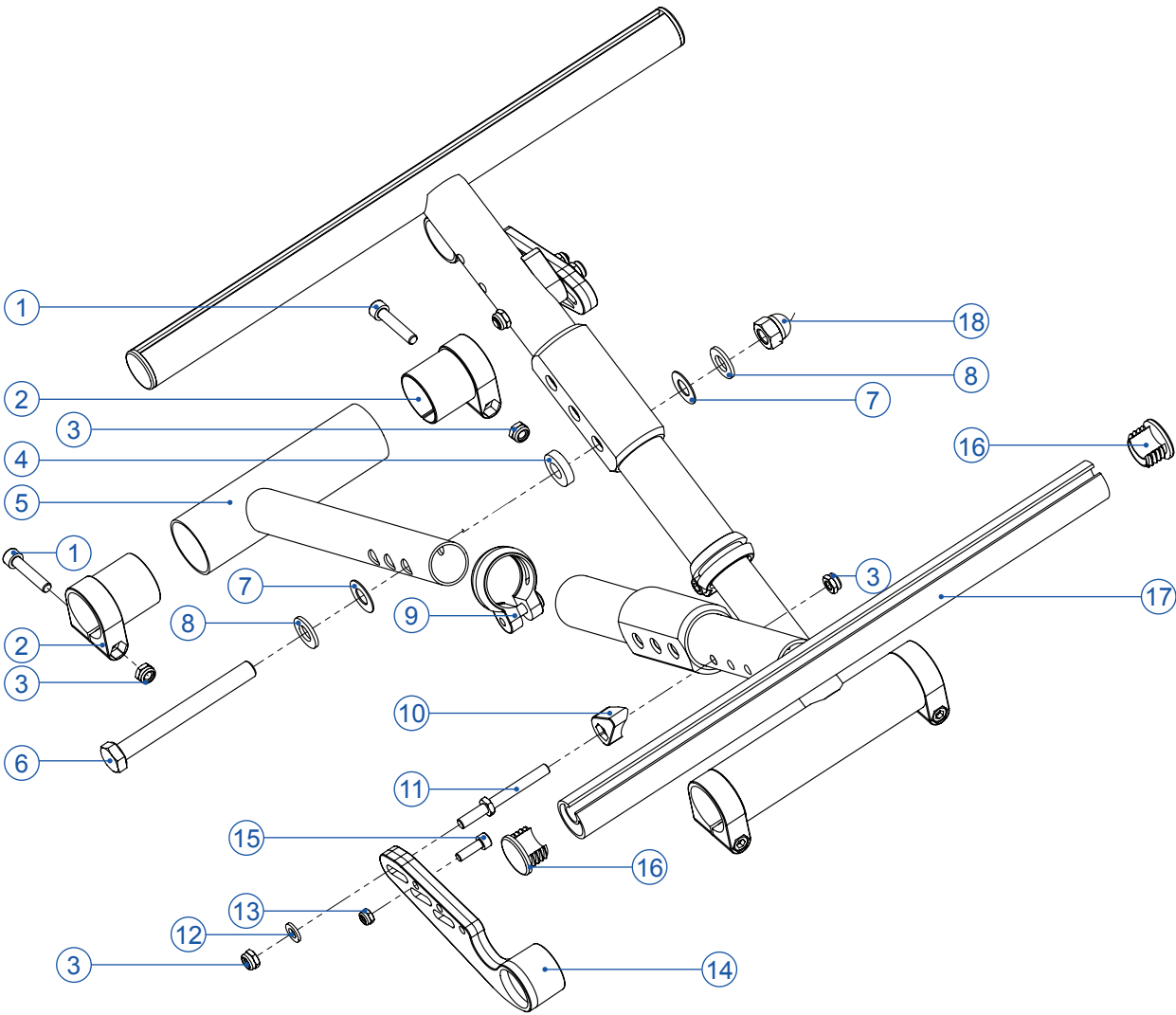


Position	Article number	Article description	Quantity
1	1021-0630	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x30	4 Pc.
2	3224-1293	Cross brace bushing	4 Pc.
3	1042-0006	Hexagon nut self-locking zp, DIN/ISO 985, M6	8 Pc.
4	1113-0060	Washer PA black, 25x10,2x5	1 Pc.
5	3225-1210	Base cross brace size 1 WC	2 Pc.
6	1010-1095	Hexagon head screw zp, DIN/ISO 931, M10x95	1 Pc.
7	1113-0056	Washer PA natural 25x10, 2x1	2 Pc.
8	1051-0010	Washer zp, DIN/ISO 125/7089, A10,5	2 Pc.
9	1129-0010	Saddle clamp D29,8	2 Pc.
10	3224-1296	Guide cam	2 Pc.
11	3224-1297-008	Cam pin	2 Pc.
12	1051-0006	Washer zp, DIN/ISO 125/7089, A6,4	2 Pc.
13	1042-0005	Hexagon nut self-locking zp, DIN/ISO 985, M5	2 Pc.
14	3224-1295	Guide lever	2 Pc.
15	1021-0516	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M5x16	2 Pc.
16	3224-1224	Ribbed insert for seat rail	4 Pc.
17	3224-1230	Cross brace top size 1, SD 30	2 Pc.
	3224-1235	Cross brace top size 1, SD 35	2 Pc.
	3224-1240	Cross brace top size 1, SD 40	2 Pc.
18	1044-0010	Hexagon domed cap nut self-locking zp, DIN 986, M10	1 Pc.

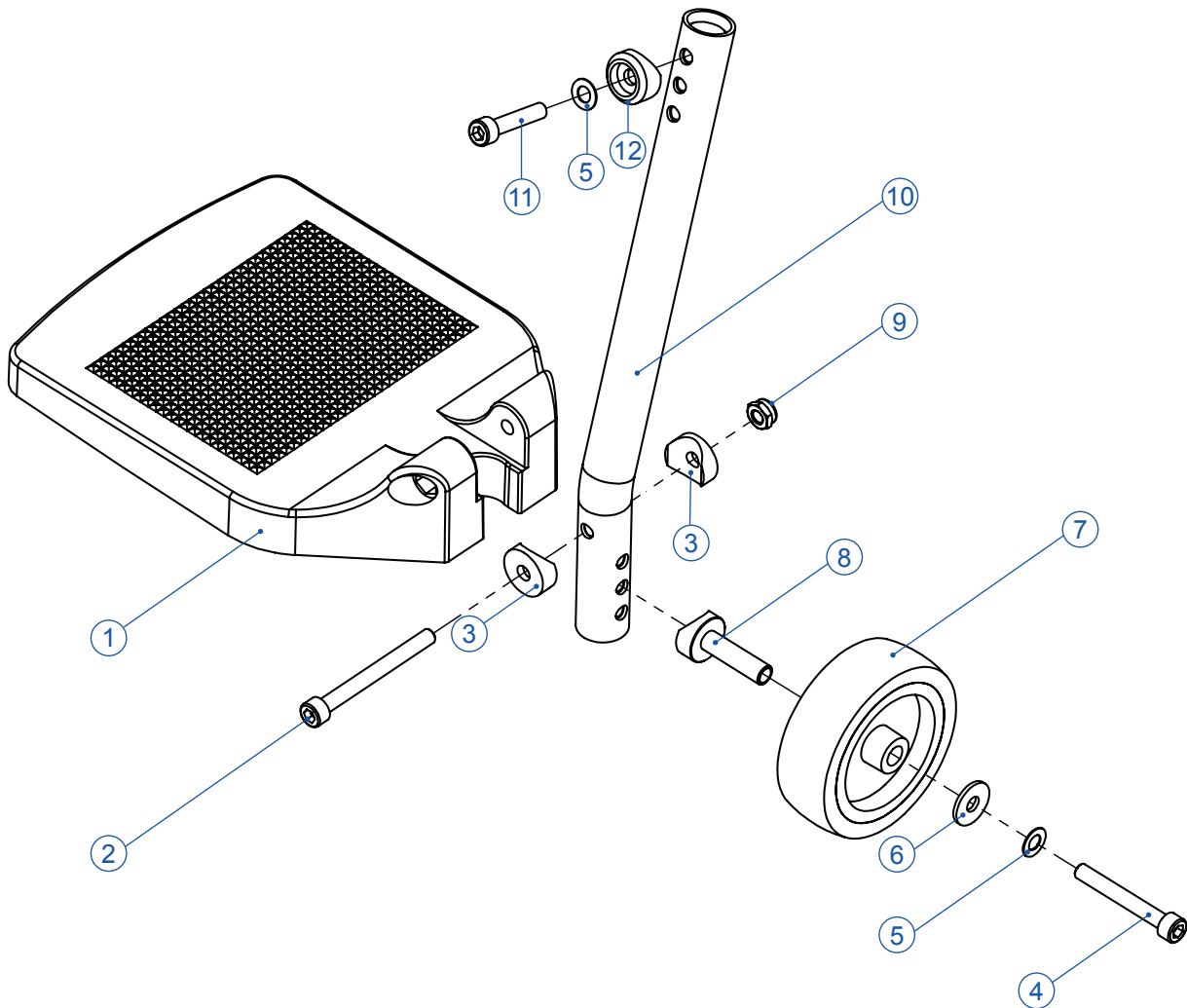


Position	Article number	Article description	Quantity
1	7000-0106	Footboard standard, L145xW150	1 Pc.
	7000-0107	Footboard standard, L170xW150	1 Pc.
	7000-0108	Footboard standard, L195xW150	1 Pc.
2	1021-0660	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x60	1 Pc.
3	1111-0019	Saddle washer, 20x6,4x2,5	2 Pc.
4	1021-0645	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x45	1 Pc.
5	1052-0006	Spring washer zp, DIN 137 B, B6	2 Pc.
6	1055-0006	Washer zp, DIN/ISO 9021/7093, A6,4	1 Pc.
7	1249-0009	Roll Ø75x24	1 Pc.
8	3224-1421	Roll axle	1 Pc.
9	1042-0006	Hexagon nut self-locking zp, DIN/ISO 985/10511, M6	1 Pc.
10	3225-1410	Footplate hanger WC	1 Pc.
11	1021-0630	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x30	1 Pc.
12	3224-1423-007	Saddle washer 20x12,5x6,2	1 Pc.

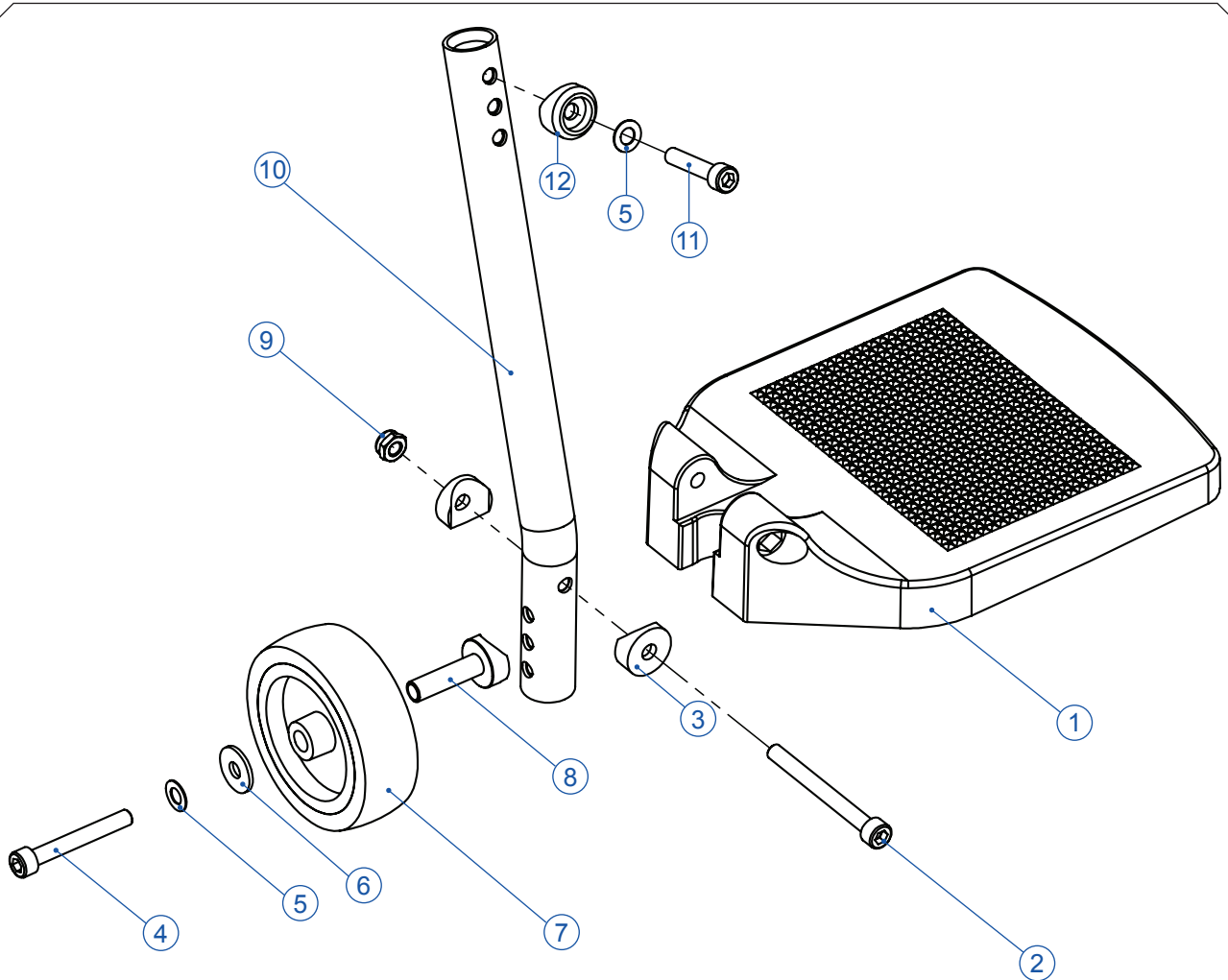


Position	Article number	Article description	Quantity
1	7000-0106	Footboard standard, L145xW150	1 Pc.
	7000-0107	Footboard standard, L170xW150	1 Pc.
	7000-0108	Footboard standard, L195xW150	1 Pc.
2	1021-0660	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x60	1 Pc.
3	1111-0019	Saddle washer, 20x6,4x2,5	2 Pc.
4	1021-0645	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x45	1 Pc.
5	1052-0006	Spring washer zp, DIN 137 B, B6	2 Pc.
6	1055-0006	Washer zp, DIN/ISO 9021/7093, A6,4	1 Pc.
7	1249-0009	Roll Ø75x24	1 Pc.
8	3224-1421	Roll axle	1 Pc.
9	1042-0006	Hexagon nut self-locking zp, DIN/ISO 985/10511, M6	1 Pc.
10	3225-1410	Footplate hanger WC	1 Pc.
11	1021-0630	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x30	1 Pc.
12	3224-1423-007	Saddle washer 20x12,5x6,2	1 Pc.

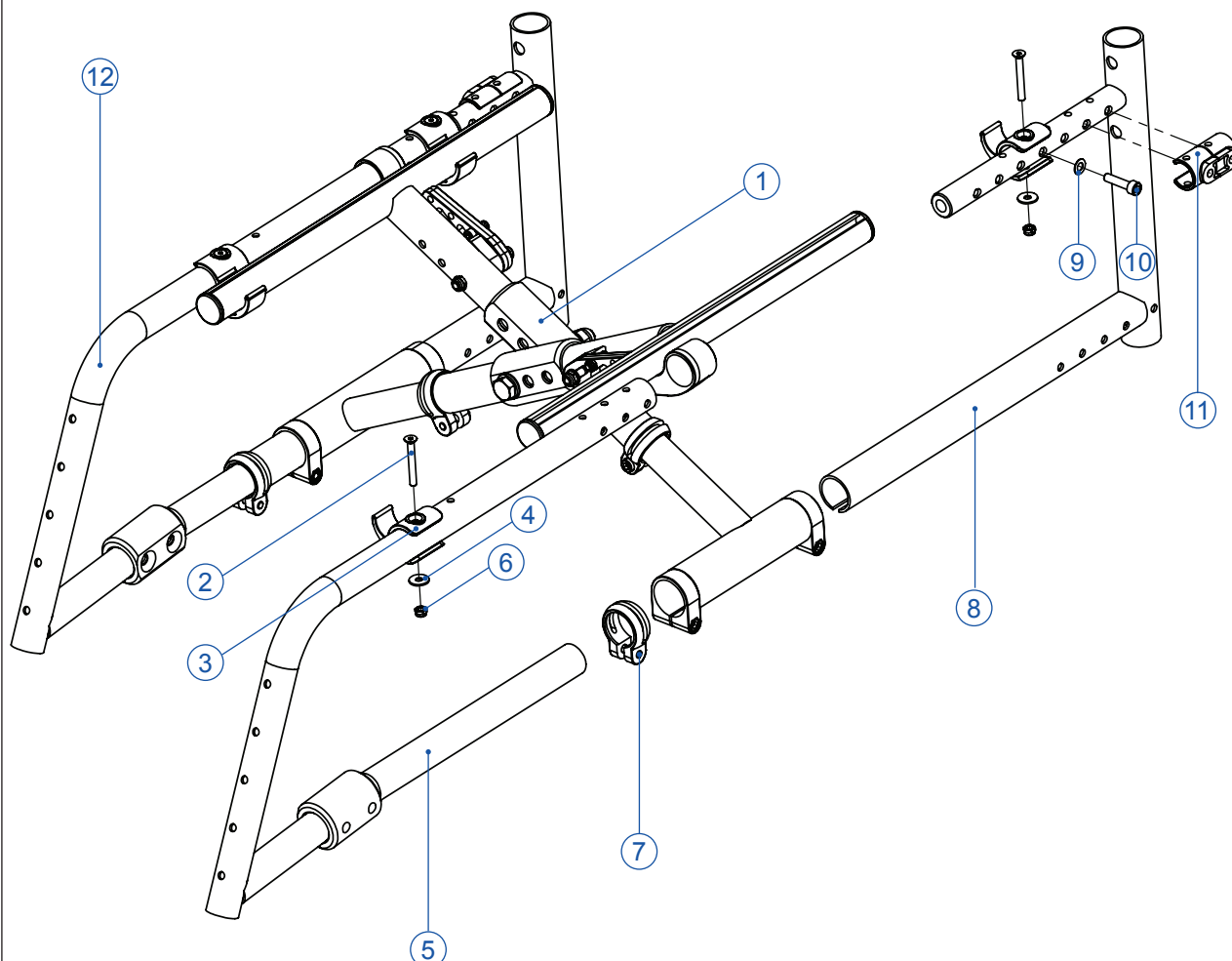

Position	Article number	Article description	Quantity
1	3225-1200	Cross brace size 1 assy	1 Pc.
2	1024-0540	Hexagon socket countersunk head screw zp, DIN/ISO 7991, M5x40	4 Pc.
3	3224-1292	Seat rail rest	4 Pc.
4	1055-0005	Washer zp, DIN/ISO 9021/7093, A5,3	4 Pc.
5	3225-1130	Side frame left size 1 WC	1 Pc.
6	1042-0005	Hexagon nut self-locking zp, DIN/ISO 985/10511, M5	4 Pc.
7	1129-0010	Saddle clamp D29,8	2 Pc.
8	3225-1110	Frame base size 1 WC	2 Pc.
9	1052-0006	Spring washer zp, DIN 137 B, B6	2 Pc.
10	1021-0625	Hexagon socket head cap crew zp, DIN/ISO 912/4762, M6x25	2 Pc.
11	3224-1152	Spacer 25/50	2 Pc.
12	3225-1140	Side frame right size 1 WC	1 Pc.

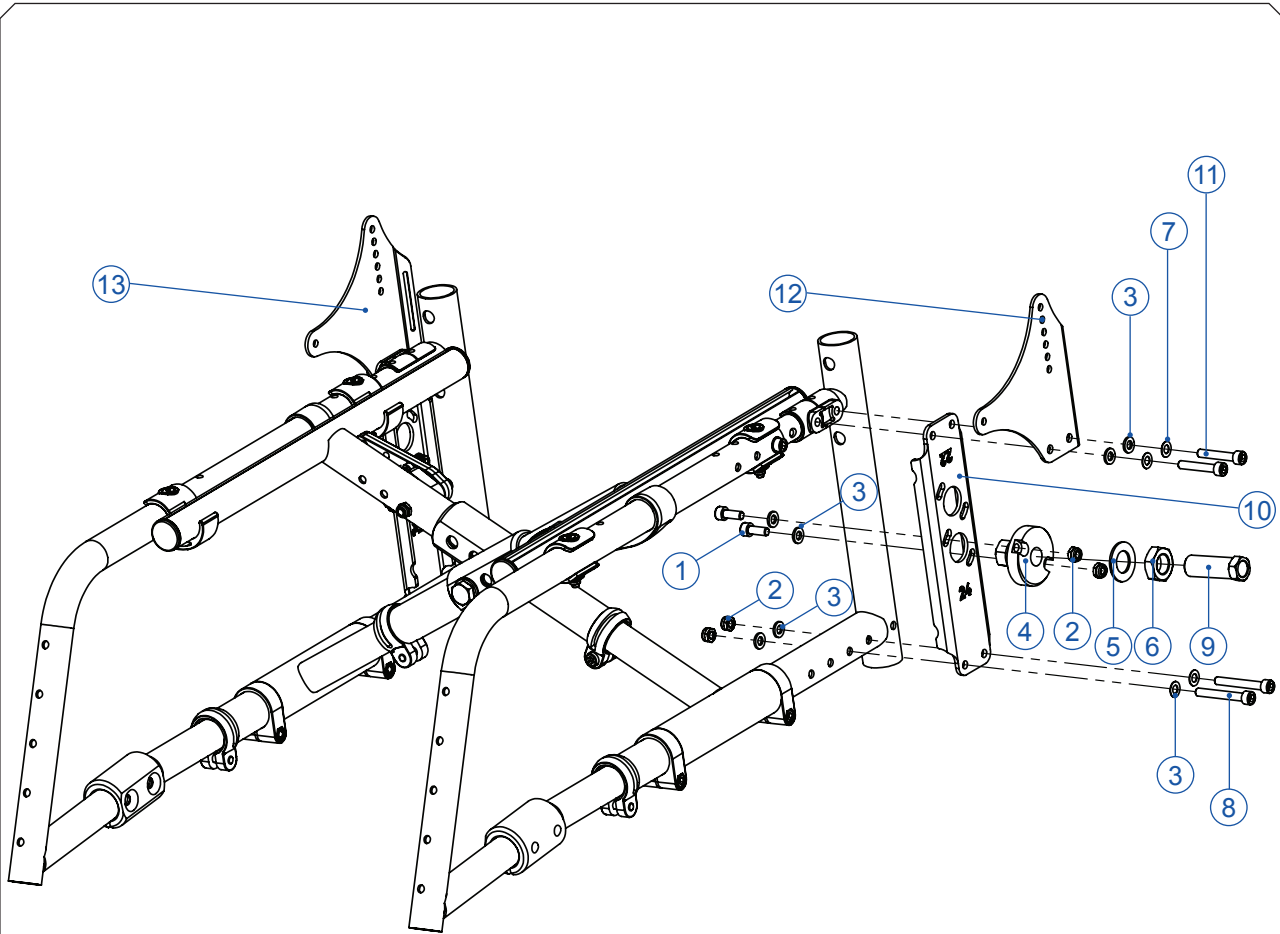


Position	Article number	Article description	Quantity
1	1021-0616	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x16	4 Pc.
2	1042-0006	Hexagon nut self-locking zp, DIN/ISO 985/10511, M6	8 Pc.
3	1051-0006	Washer zp, DIN/ISO 125/7089, A6,4	16 Pc.
4	3224-1162-006	Chamber adapter 3°	2 Pc.
5	1052-0016	Spring washer zp, DIN 137 B, B16	2 Pc.
6	1040-0016	Hexagon nut zp, DIN 936, M16x1	2 Pc.
7	1052-0006	Spring washer zp, DIN 137 B, B6	4 Pc.
8	1021-0645	Hexagon socket head cap, screw zp, DIN/ISO 912/4762, M6x45	4 Pc.
9	1205-0059	Adapter for quick release axle, 12/50mm	2 Pc.
10	3224-1151	Adapter plate size 1	2 Pc.
11	1021-0635	Hexagon socket head cap, screw zp, DIN/ISO 912/4762, M6x35	4 Pc.
12	3225-1512	Fender bracket left	1 Pc.
13	3225-1522	Fender bracket right	1 Pc.

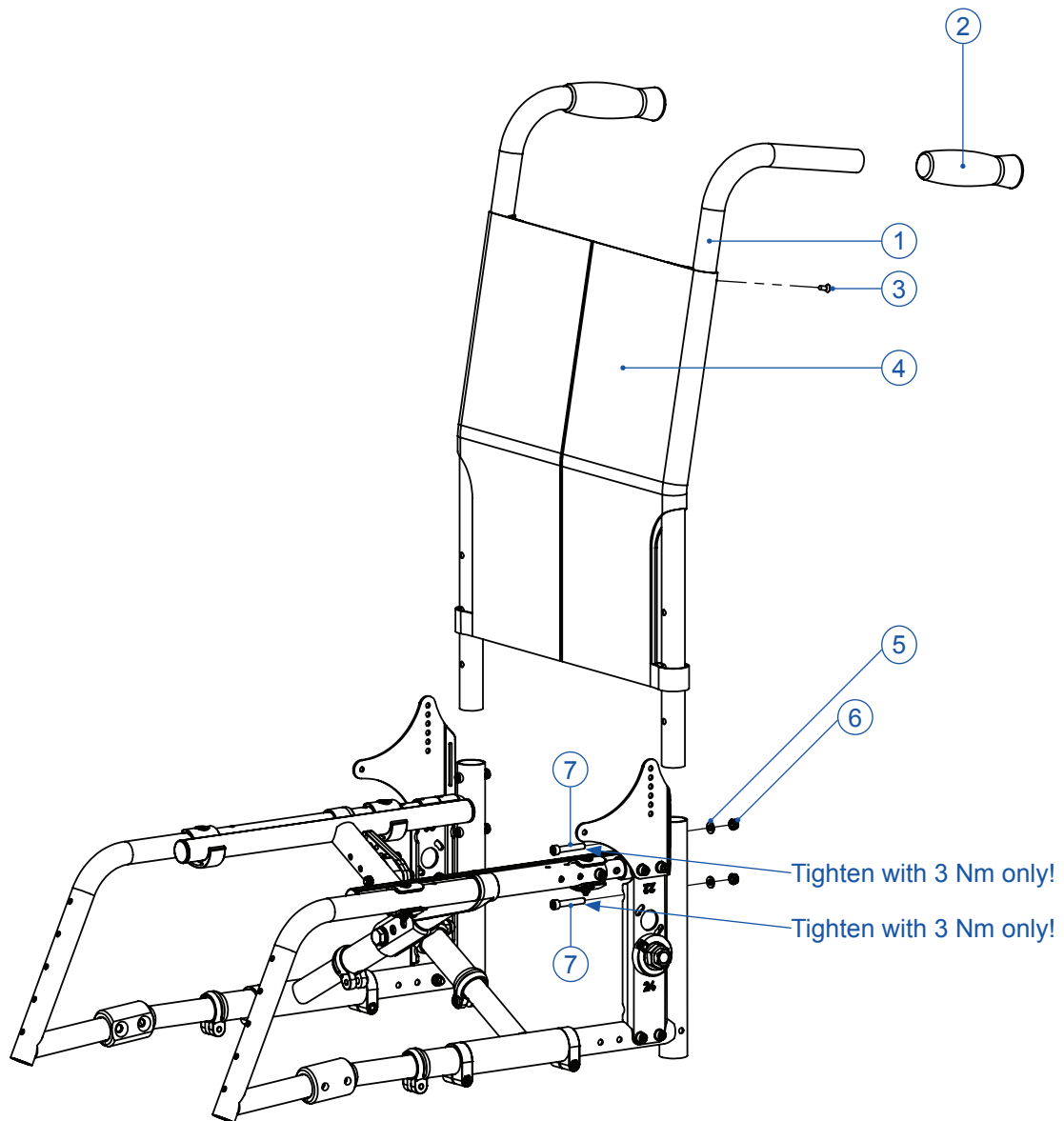


For setting back height and handle height please see chapter **"1 - settings"** page **10**


Position	Article number	Article description	Quantity
1	3225-1311	Back cane size 1	2 Pc.
2	1143-0005	Handgrip PE 125x25	2 Pc.
3	1039-0006	Cross rec. countersunk head tapping screw, ISO 7982 C, 3,9x13	2 Pc.
4	3225-2201	Back sling SW 30	1 Pc.
	3225-2202	Back sling SW 35	1 Pc.
	3225-2203	Back sling SW 40	1 Pc.
5	1051-0006	Washer zp, DIN/ISO 125/7089, A6,4	4 Pc.
6	1042-0006	Hexagon nut self-locking zp, DIN/ISO 985/10511, M6	4 Pc.
7	1021-0635	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x35	4 Pc.

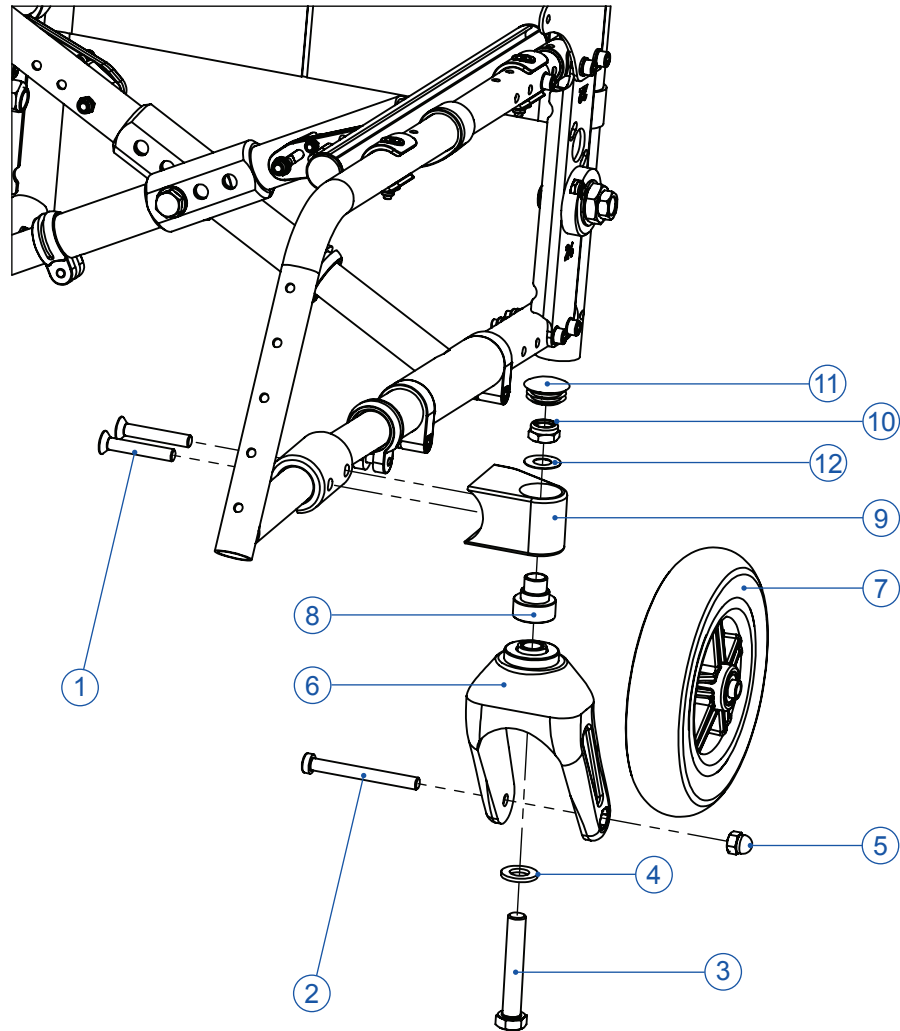

Position	Article number	Article description	Quantity
1	1024-0850	Hexagon socket countersunk head screw zp, DIN/ISO 7991, M8x50	4 Pc.
2	1022-0875	Hexagon socket head cap screw zp, DIN/ISO 7984, M8x75	2 Pc.
3	1012-1270	Hexagon head screw zp, DIN/ISO 931/4014, M12x70, SW 19	2 Pc.
4	1051-0012	Washer zp, DIN/ISO 125/7089, A13	2 Pc.
5	1044-0008	Hexagon domed cap nut selflocking, zp, DIN/ISO 986/-M8	2 Pc.
6	1281-0018	Caster fork LG-K-200-15/12	2 Pc.
7	1221-1950	PU wheel, 190x50 / 7,5"x2"	2 Pc.
8	3224-1173	Caster fork bushing	2 Pc.
9	3224-1172-007	Caster fork housing	2 Pc.
10	1042-0012	Hexagon nut self-locking zp, DIN/ISO 985/10511, M12	2 Pc.
11	1141-0016	Ribbed insert SFL 30x0,8-2.5	2 Pc.
12	1059-0013	Distance washer 12x24x0,5	1 Pc.

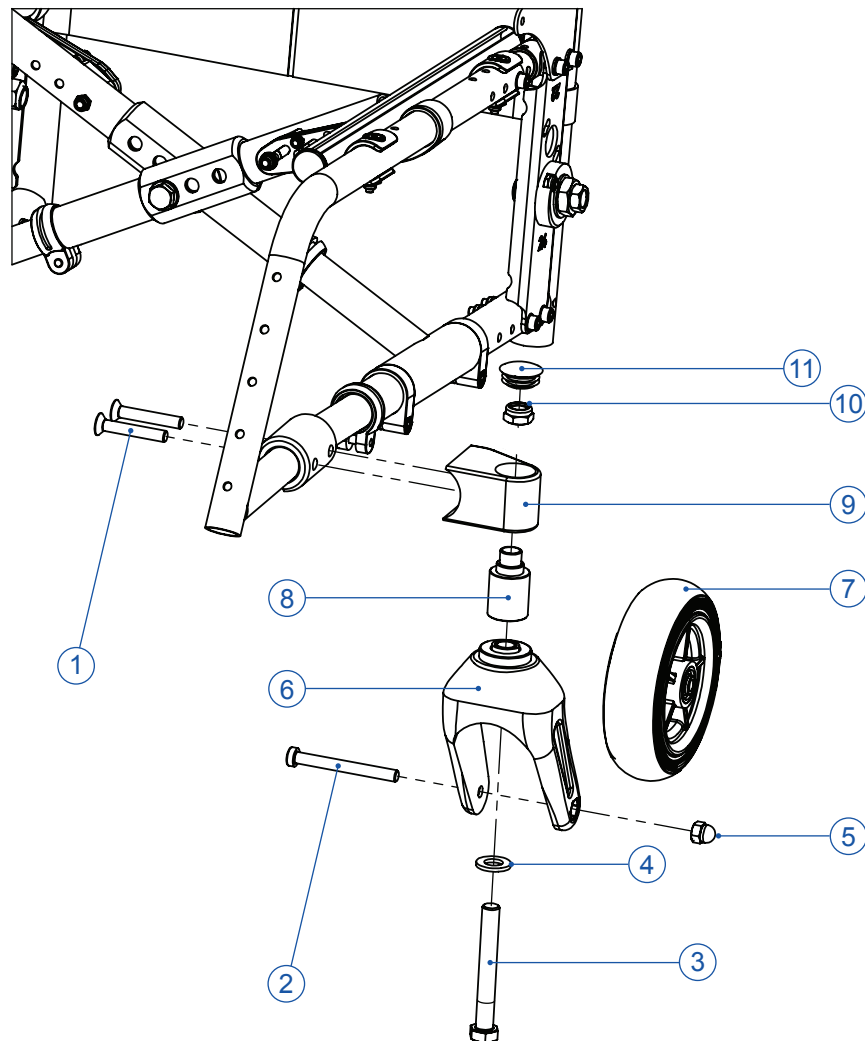


Position	Article number	Article description	Quantity
1	1024-0850	Hexagon socket countersunk head screw zp, DIN/ISO 7991, M8x50	4 Pc.
2	1022-0875	Hexagon socket head cap screw zp, DIN/ISO 7984, M8x75	2 Pc.
3	1012-1290	Hexagon head screw zp, DIN/ISO 931/4014, M12x90, SW 19	2 Pc.
4	1051-0012	Washer zp, DIN/ISO 125/7089, A13	2 Pc.
5	1044-0008	Hexagon domed cap nut selflocking, zp, DIN/ISO 986/-M8	2 Pc.
6	1281-0018	Caster fork LG-K-200-15/12	2 Pc.
7	1250-1441	Elastic-wheel R140x40	2 Pc.
8	3224-1176-007	Caster fork bushing, long	2 Pc.
9	3224-1172-007	Caster fork housing	2 Pc.
10	1042-0012	Hexagon nut self-locking zp, DIN/ISO 985/10511, M12	2 Pc.
11	1141-0016	Ribbed insert SFL 30x0,8-2.5	2 Pc.

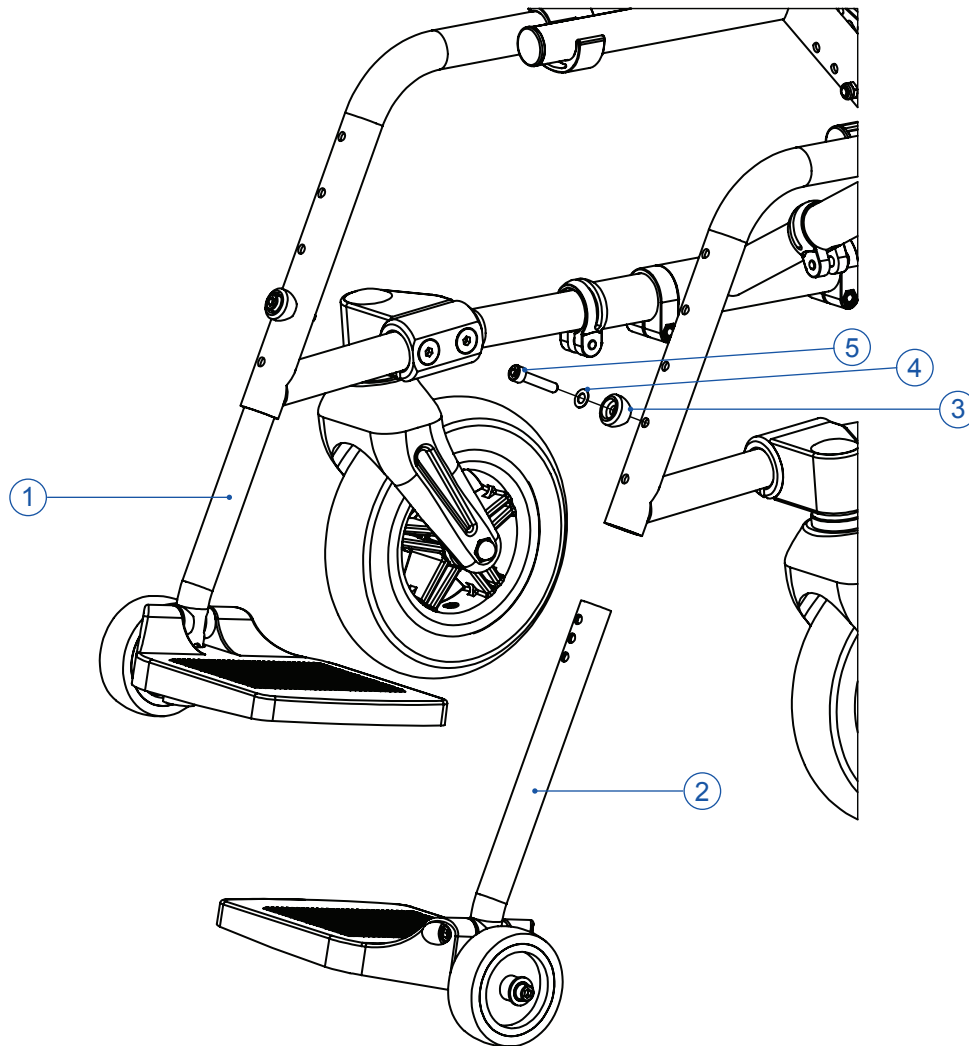


For setting lower leg length please see chapter **"1 - settings"** page 8


Position	Article number	Article description	Quantity
1	3225-1401-R	Footrest assembly size 1 30/12" right	1 Pc.
	3225-1402-R	Footrest assembly size 1 35/14" right	1 Pc.
	3225-1403-R	Footrest assembly size 1 40/16" right	1 Pc.
2	3225-1401-L	Footrest assembly size 1 30/12" left	1 Pc.
	3225-1402-L	Footrest assembly size 1 35/14" left	1 Pc.
	3225-1403-L	Footrest assembly size 1 40/16" left	1 Pc.
3	3224-1423-007	Saddle washer 20x12,5x6,2	2 Pc.
4	1052-0006	Spring washer zp, DIN 137 B, B6	2 Pc.
5	1021-0630	Hexagon socket head cap screw zp, DIN/ISO 912/4762, M6x30	2 Pc.

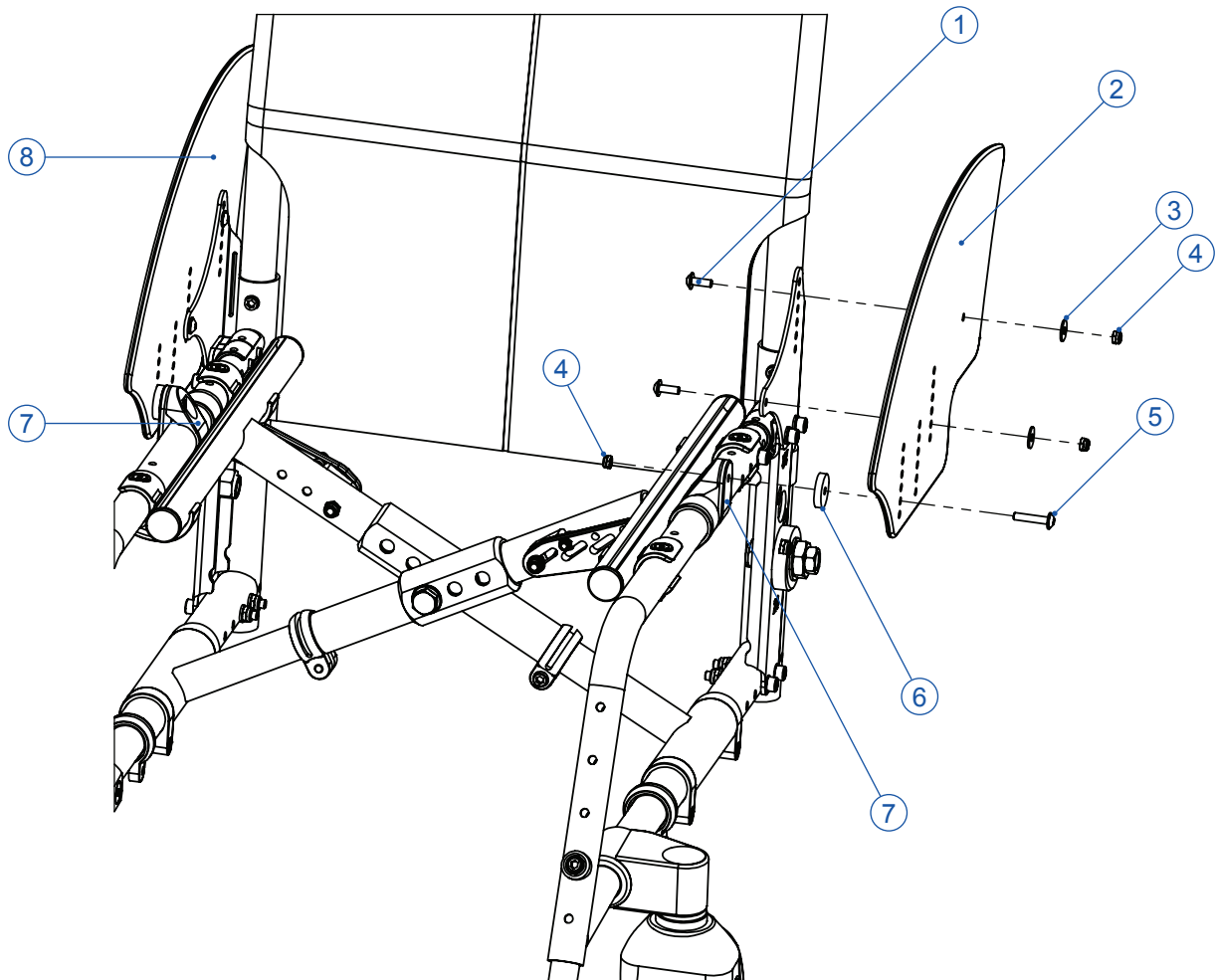


Position	Article number	Article description	Quantity
1	1030-0514	Button head socket screw with flange, M5x14	4 Pc.
2	3225-1513	Side guard 22"+24", left	1 Pc.
3	1055-0005	Washer zp, DIN/ISO 9021 A5,3	4 Pc.
4	1042-0005	Hexagon nut selflocking zp, DIN/ISO 985, M5	6 Pc.
5	1030-0525	Button head socket screw with flange, M5x25	2 Pc.
6	1113-0061	Washer PA black 24x5,3x5	2 Pc.
7	3245-3311	Gripper clamp D 25	2 Pc.
8	3225-1514	Side guard 22"+24", right	1 Pc.

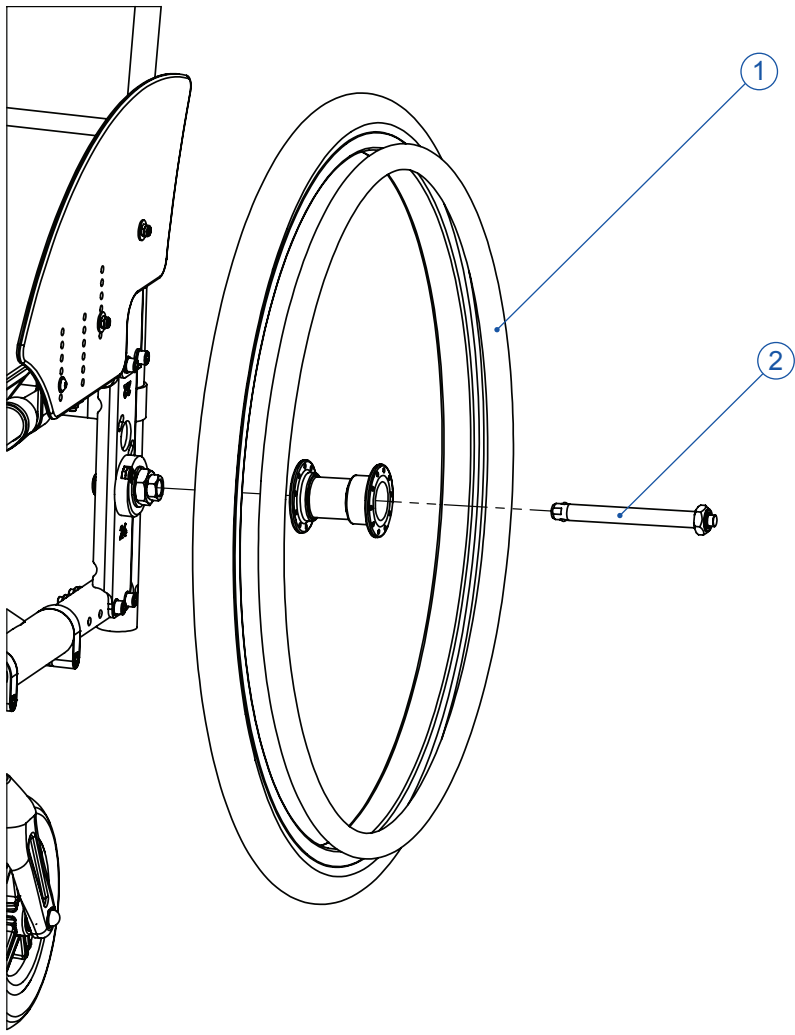

Position	Article number	Article description	Quantity
1	1221-0031	Spoked wheel 22"x1 3/8	2 Pc.
	1221-0032	Spoked wheel 24"x1 3/8	2 Pc.
2	1205-0058	Quick release axle 12x103	2 Pc.

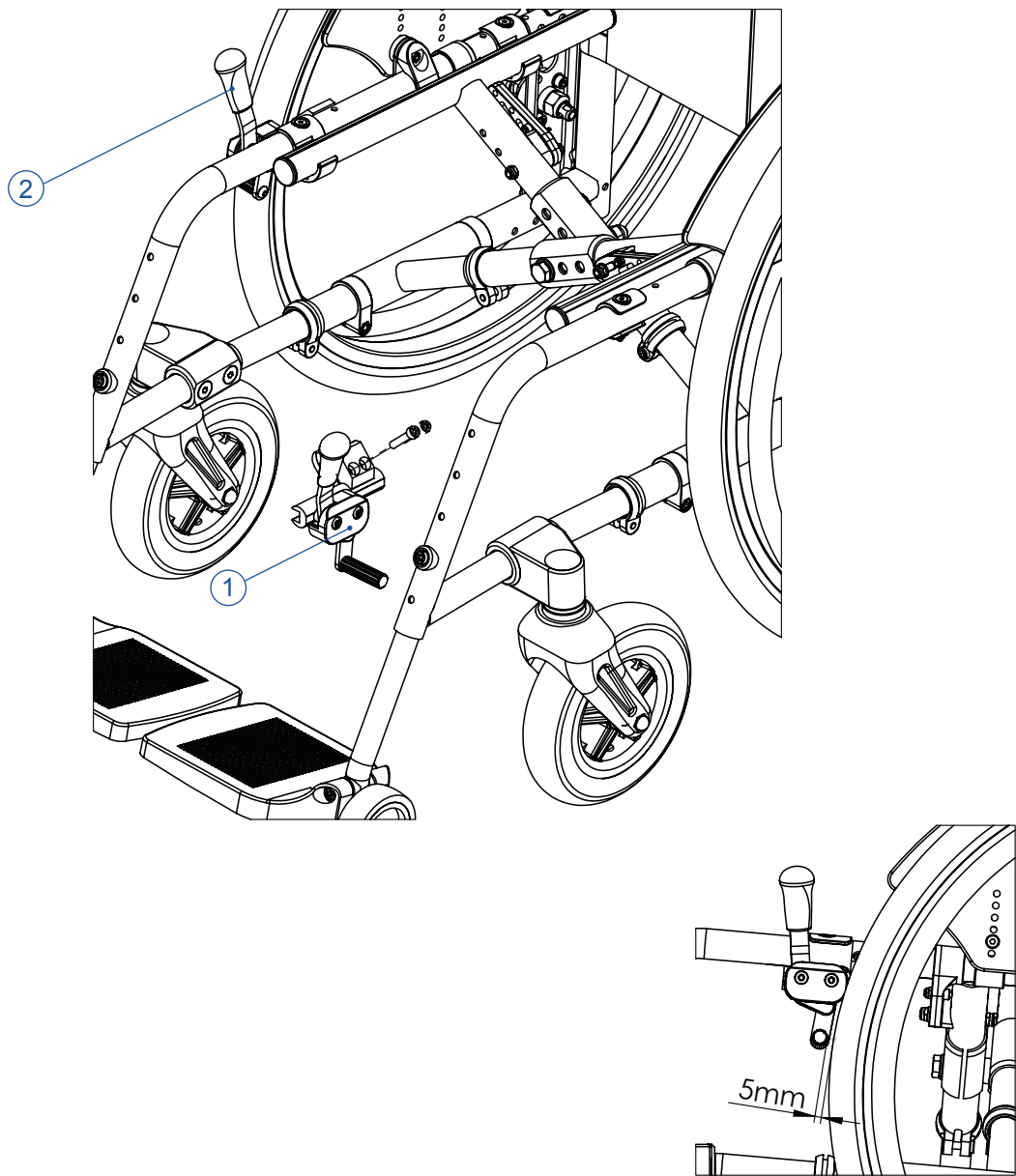

Position	Article number	Article description	Quantity
1	3224-1701	Wheel lock, left	1 Pc.
2	3224-1702	Wheel lock, right	1 Pc.

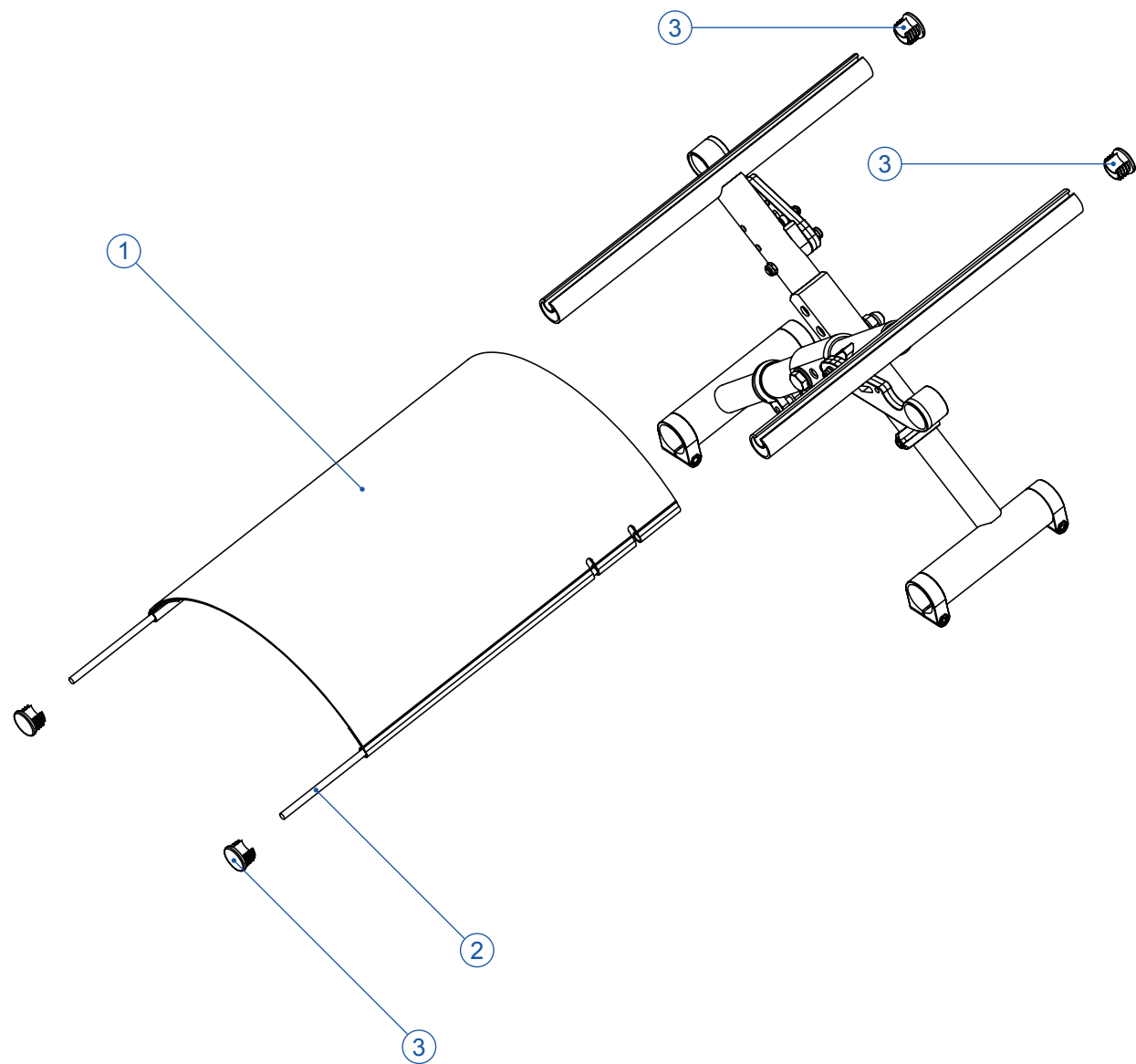

Position	Article number	Article description	Quantity
1	3225-2101	Seat sling SW 30	1 Pc.
	3225-2102	Seat sling SW 35	1 Pc.
	3225-2103	Seat sling SW 40	1 Pc.
2	3225-1229	Beading 30	2 Pc.
	3225-1239	Beading 35	2 Pc.
	3225-1249	Beading 40	2 Pc.
3	3224-1224	Ribbed insert for seat rail	4 Pc.

4 - Your notes

[illegible]

[illegible]



Rock The World.

HOGGI

Partnering together to mobilize kids.

HOGGI GmbH

 Eulerstraße 27
D-56235 Ransbach-Baumbach

 Telefon: (+49) 2623 / 92 499-0
Telefax: (+49) 2623 / 92 499-99

 info@hoggi.de

www.hoggi.de